

Established 1972 By and For the People of Love County Richard Barker, Administrator

Ambulance Emergency

911

Clinic Appointments

276-2400

Hospital Information

276-3347

Community Resources

211

Domestic Violence Help

276-2042 Hotline - 226-6424

PATIENT LIFTING



Hovermatt Air Transfer System

Hoverjack Air Patient Lift



Left - clockwise from top: Marneta Dorsey, Marie Ross, Gay Galano, and Danny Dvorak.

	What Is It? Year Acquired:	Hovermatt Moving Air Transfer Technology System 2015
-	Use:	Allows one or two nurses to easily transfer patients from stretcher to bed or raise them from the floor if they have fallen.
		Air forced through tiny holes causes the mat to hover, reducing the load being moved to 10 percent of the patient's weight.
	How It Works:	Hovermatt: The nurse rolls the patient onto the deflated Hovermatt and buckles the safety straps. The mat is aired up using the vacuum hose and electrical air supply unit. If this is a bed-to-bed transfer, the nurse uses strap handles on the side to slide the hovering mat onto the adjacent surface. In a floor-to-bed situation, the nurse first slides the mat onto the Hoverjack (see below).
		Hoverjack: The patient is placed on top of the deflated Hoverjack using the Hovermatt. The Hoverjack is then inflated, one chamber at a time, until the hovering mat reaches bed or stretcher height.

We're YOUR Hospital, Clinic and EMS Mercy Health/Love County

The Small-Town Hospital with the Big Heart

300 Wanda - Marietta, OK 73448

www.mercyhealthlovecounty.com